

Instructions for **Nitradine Cleaning Tablets**

Please watch our tutorials, read these instructions & terms and conditions before attempting the impression.

*Please Note: Do not attempt to take your impressions on a hot day or in overly warm conditions.

1

Rinse The Impression

Rinse the impression you've created under running water.

2

Place the Tablet in Water

Place the impression in a bowl of warm water ensuring that it is fully immersed and immediately place 1 tablet in the water. Soak for a minimum of 15 mins.

3

Rinse Well

Remove the impression from the solution and rinse well underwater.

If your tablet is broken the effectiveness and usage of the product will not be affected (provided the whole tablet is used)